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
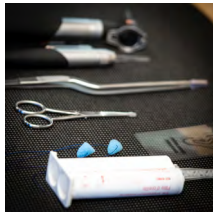
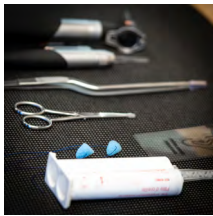
IMPRESSION GUIDE FOR ACOUSTICIANS

THE PERFECT IMPRESSIONS FOR IN-EARS


In order to ensure an optimal fit of the custom-fitted in-ears, some important points must be observed during the moulding process. Unfortunately, no guarantee of fit and sound can be given if these points are not observed. Customers have to bear the costs for the impression themselves.

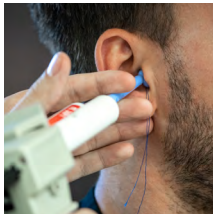
Please use an impression material with the highest possible impression precision. The impression silicones **HL SOFT**, **HL SPEED** or **HL FLOW** from Hörluchs® are ideally suited. The impression must show the concha completely and without bubbles. It is also important to ensure that the anthelix, crus helicis, tragus, antitragus, incisura intertragica and Anterior are completely imaged.

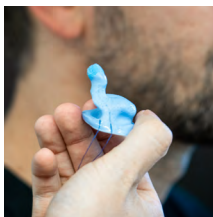The shells of the in-ears must sit absolutely tight in the ear. Make sure that the impression is taken dynamically, i.e. with chewing movements, e.g. chewing gum.

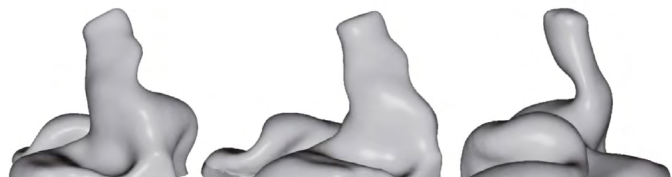The length of the moulded ear canal must represent at least the **second ear canal bend**. The impression must not be edited or trimmed in any way.

EU Customers: Please send the impressions to

Hörluchs Hearing GmbH & Co. KG
Wareneingang
Friedrich-Köchert-Str. 4
91217 Hersbruck
DE - Germany

Swiss Customers: Please send the impressions to

Hörluchs Scanstation
Mike Sonnenberg
Schulstr. 21
4402 Frenkendorf
CH - Switzerland

Please do not forget to include your exact contact information and order number.